



BOWL MIXER


Bowl Mixer for Bone Cement

General Description

Bowl - Mixer system is a convenient and efficient vacuum mixing and application system for orthopaedic bone cement. It has an easy to use design that makes possible the preparation of perfectly homogenous and void-free bone cements. Ultra-Mixer is suitable for mixing bone cements of various viscosities.

Package Contents

- ✓ 1 Mixing Bowl
- ✓ 1 Air Filter
- ✓ 1 Vacuum hose
- ✓ 1 Scraper

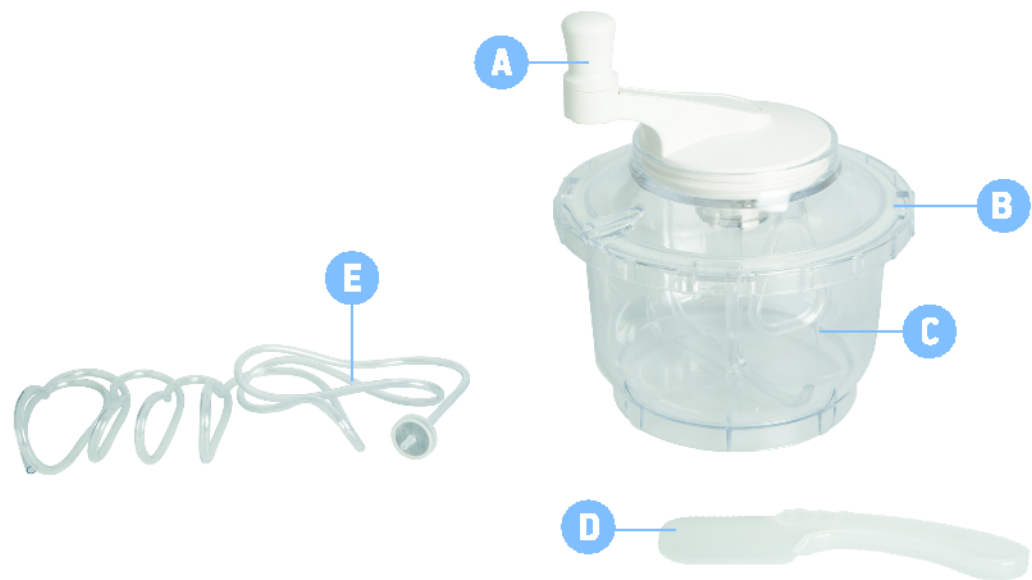

Instructions for Use

1. Before opening the package check that it is undamaged and that it contains all the items.
2. Peel-open the pack, remove all of the items and place them in the sterile surgical field.
3. Place the mixing bowl on a flat surface and attach the air filter on the mixer cap while the other end of the hose of the air filter is connected to the vacuum tube. **B E**
4. First, pour in the bowl the monomer liquid and then the cement powder. **C**
5. Close securely the mixing cap by rotating it clockwise and ensure that the vacuum source is working.
6. Mix the cement by rotating the handle of mixer cap. Stop the mixing when the cement starts to homogenizing, according to the instructions of cement manufacturer. **A**
7. Apply the cement with the use of the scraper. **D**

Warnings

- If the package of the Bowl Mixer system is damaged or if the expiry date has elapsed the system must no longer be used.
- The device should be disposed safely and in accordance with all current national and international waste disposal directives.
- Store in a cool dry place.
- Single use only! DO NOT Re-sterilize!



STERILE EO
Do not re-sterilise



1 Kings Avenue London N21 3 NA, UK
t: +44 (0) 208 3707764
f: +44 (0) 208 3707292
website: www.vadinimplants.com
e-mail: Info@vadinimplants.com