

General Description

The 2500C Pulse Lavage system is an efficient, high-pressure irrigation system. It is a BUILD IN battery pack that may be connected to several tips as to facilitate any specific application.

Indications

• In Joint Replacements

The 2500C Pulse Lavage system uses pulsed jets of saline to remove blood and debris from the cancellous bone and to prepare an enhanced implant bed. The thorough cleaning of the bone canal also reduces the risk for fat embolism.

• In Wound Care


The high-pressure irrigation system is used to thoroughly remove necrotic tissues, bacteria, and foreign materials from wounds, while the harm caused to healthy tissues is minimal.

Package Contents

- ✓ 1 Handpiece
- ✓ 1 Small cone tip (ST01 Z)
- ✓ 1 Coaxial Canal tip (CT02 Z)

Product Characteristics

- DC 10.5V / AA x 7 cells Alkaline Battery.
- 3 meter long tube.
- 1000ml/m High speed flow rate.
- 700ml/m Low speed flow rate.
- Up to 15 PSI pressure.

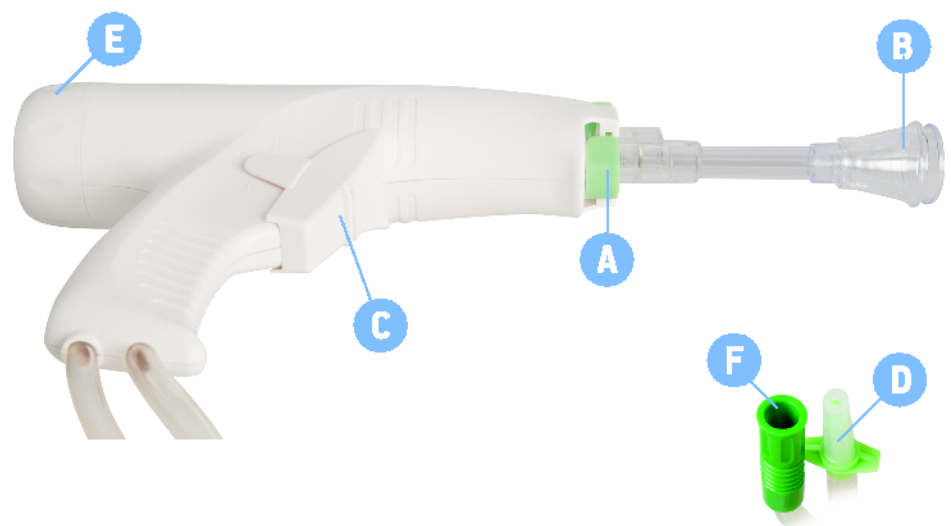


Instructions for Use

1. Before opening the package check that it is undamaged and that it contains all the items.
2. Peel-open the pack, remove all of the items and place them in the sterile surgical field.
3. Move the Locking Ring (A) to the upper position.
4. Select the appropriate tip, attach it to the handpiece and accordingly secure it into place by moving the Locking Ring at the lower position. (If the Intra Medullary Tip is to be used, the white cap should be removed before use) (B)
5. Place the hose irrigation tip (D) to the irrigation source.
6. Place the hose suction tip (F) to the suction source. The suction source and the handpiece should not have considerable height difference; otherwise they liquid may escape from the tip.
7. Remove the protective lid from the Trigger (C)
8. According to the application the Trigger may be either pressed in the upper position for high speed liquid flow or in the lower position for low speed liquid flow. (Note: If the pulse lavage system is used for first time initially press the Trigger in the upper position for high speed flow).
9. The suction and irrigation hoses carry clamps that can be used in the event of liquid leakage.

Battery Removal

The battery is removed from the handpiece by rotating the cap counter-clockwise and pulling the battery pack out. (E)

TIPS ACCESSORY PAGE

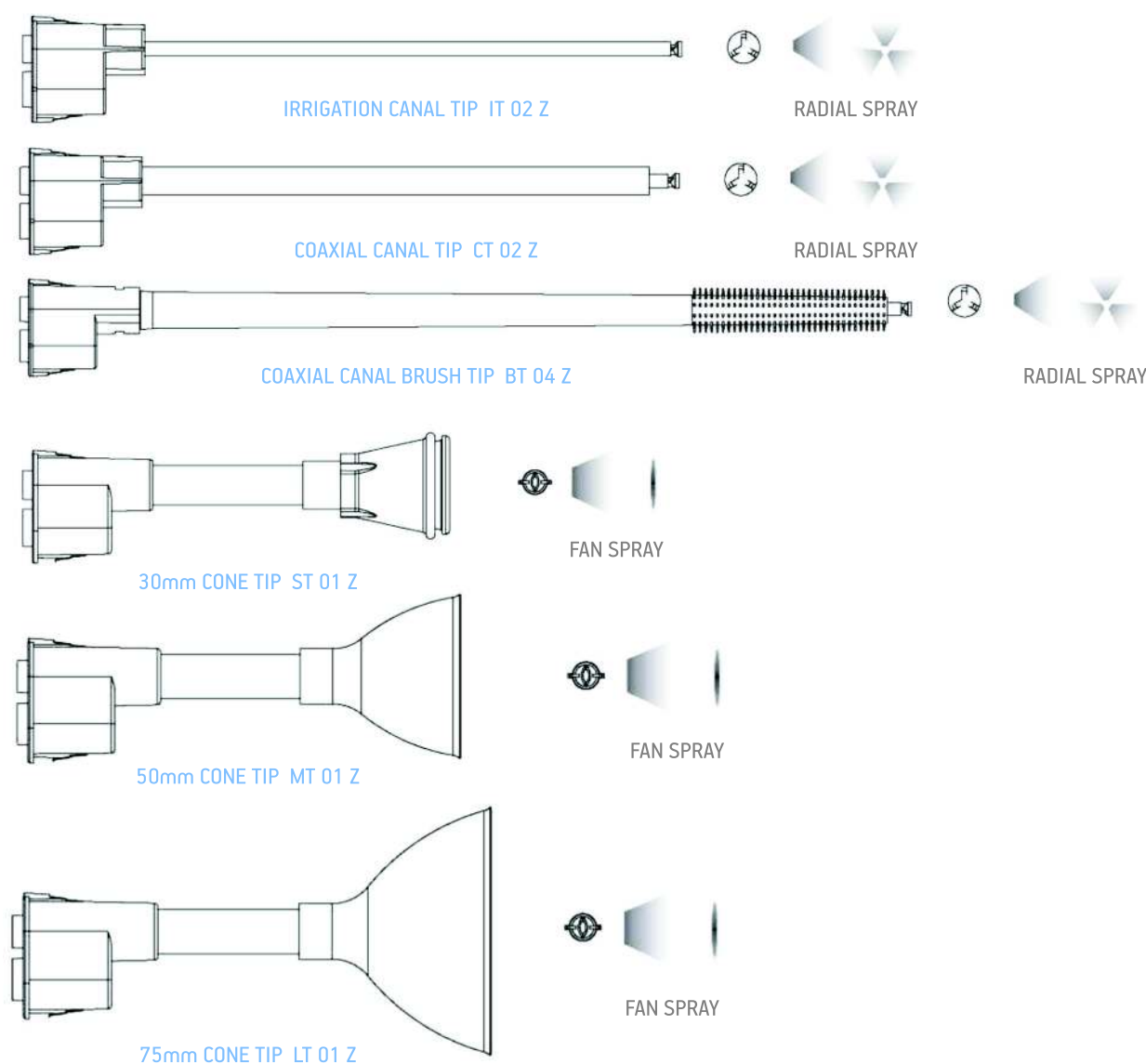

Warnings

- If the package of the 2500C Pulse Lavage system is damaged or if the expiry date has elapsed the system must no longer be used.
- The device should be disposed safely and in accordance with all current national and international waste disposal directives.
- Store in a cool dry place.
- Single use only! DO NOT Re-sterilize!



STERILE EO
Do not re-sterilise



1 Kings Avenue London N21 3 NA, UK
t: +44 (0) 208 3707764
f: +44 (0) 208 3707292
website: www.vadinimplants.com
e-mail: info@vadinimplants.com

Literature

- Sierra RJ, Timperley JA, Gie GA. Contemporary cementing technique and mortality during and after Exeter total hip arthroplasty. J Arthroplasty. 2009 Apr;24(3):325-32.
- Norton MR, Eyres KS. Irrigation and suction technique to ensure reliable cement penetration for total knee arthroplasty. J Arthroplasty. 2000 Jun;15(4):468-74.
- Breusch SJ, Norman TL, Schneider U, et.al. Lavage technique in total hip arthroplasty: jet lavage produces better cement penetration than syringe lavage in the proximal femur. J Arthroplasty. 2000 Oct;15(7):921-7.
- Schlegel UJ, Siewe J, Delank KS, Eysel P, Püschel K, Morlock MM, de Uhlenbrock AG. Pulsed lavage improves fixation strength of cemented tibial components. Int Orthop. 2011 Aug;35(8):1165-9